Septicare®

Septicare® - II - Bond Express

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Usage Instructions

You will need : 1 gun, 1 PU resin cartridge, 1 mixing tip, 1 wood block

1. Clean the claw.
2. Place the clippers on «shoulders» of the cartridge and cut.
3. Attach the mixing tip.
4. Dispense the adhesive to ensure that both sides are flowing.
5. Apply the mixed adhesive on to the block.
6. Place the block.
7. Hold the block for 30 seconds. Be sure to leave 3 mm thickness of the adhesive between the block and the claw.
8. The animals weight can be beared after 3 minutes and released.

Properties

The 2-components adhesive Septicare-II-Bond Express does not require a separate mixing process and can be applied immediately after preparation to the claw. The light-pasty viscosity prevents the spilling out of the adhesive on the claw and block. The processing time of **20-30 seconds** and a curing time of about **2-3 minutes** enables a quick claw treatment and ensures no stress for the animal.

Advice

Prepare the claw well.

After the application leave the mixing tip on the cartridge until the next use, where you only have to change the mixing tip. The exerted pressure must not be too high when pressing the block on to the claw, since the resulting adhesive layer will be too thin and no optimum adhesive bond can be formed.

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